


AeroChamber Plus Flow-Vu®

Anti-Static Valved Holding Chamber
MOUTHPIECE / LARGE MASK

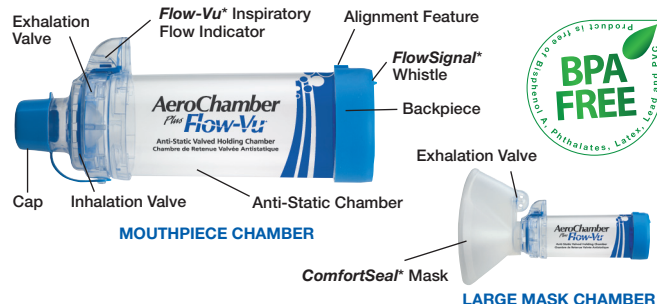

INTENDED USE

This chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to your lungs as prescribed by your healthcare provider. If you have questions about the performance or use of this product, please contact your healthcare provider.

HOW TO USE YOUR NEW CHAMBER

This chamber can be used directly out-of-package. Before use, ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times.

1 Before use, carefully examine the chamber. Replace immediately if any defect is noticed.	2 Remove cap(s) from inhaler and chamber.	3 Shake the inhaler immediately before use as per the instructions supplied with it.	4 Insert the inhaler into the Backpiece of the chamber.	5 MOUTHPIECE LARGE MASK	6 MOUTHPIECE Breathe out gently and press the inhaler at the beginning of a slow inhalation. Use the Flow-Vu® Indicator to assist in the coordination of this step. Breathe in slowly and deeply through the mouth until a full breath has been taken. Hold breath for 5 – 10 seconds, if possible. Otherwise, keep lips tight on the mouthpiece breathing normally 2 – 3 times through the chamber after inhaler is pressed. SLOW DOWN if you hear the FlowSignal® Whistle sound. It means you are inhaling too quickly. Administer one (1) puff at a time.	LARGE MASK Breathe out gently and press the inhaler at the beginning of a slow inhalation. Use the Flow-Vu® Indicator to assist in the coordination of this step. Maintain seal for 5 – 6 breaths after inhaler is pressed. SLOW DOWN if you hear the FlowSignal® Whistle sound. It means you are inhaling too quickly. Administer one (1) puff at a time.	7 Follow instructions supplied with the inhaler on how long to wait before repeating steps 3 – 6.
---	--	---	--	---	---	---	--

CLEANING INSTRUCTIONS

This chamber can be used right out-of-package and then cleaned weekly.

1 Remove the Backpiece. To detach the Frontpiece, twist chamber as shown. Remove mouthpiece cap (if applicable).	2 Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water.	OR Place parts in top rack of dishwasher. Ensure product is securely placed face up as pictured. Run the dishwasher on a normal or light cycle and avoid heated dry. Do not boil or sterilize.	3 Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly.	4 To reassemble, fit the Frontpiece on the end of the chamber and twist firmly until securely locked into position. For mouthpiece models, the protective cap should always be placed on the mouthpiece when the chamber is not in use.	5 Center the Alignment Feature on the Backpiece with the Flow-Vu® Indicator, as shown. Press firmly to attach the Backpiece.
---	--	---	--	--	--

Notes:

- Product should be replaced after 12 months of use. Environmental conditions, storage and frequency of use affect product lifespan.
- This product contains no latex.
- Do not share this medical device.
- When not in use, store in a suitable container.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.

- If cleaning in a dishwasher use a rinse aid.

⚠ Cautions:

- Do not leave the chamber unattended with children. This is not a toy.
- Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

Manufactured by:



Trudell Medical International®

725 Third Street, London, Ontario,
Canada N5V 5G4
customerservice@trudellmed.com
+1-866-510-0004
www.trudellmed.com

AeroChamber Plus® Flow-Vu® Chamber complies with CSA Standard Z264. 1-02.
101868-001 Rev D. * trade-marks and registered trade-marks of Trudell Medical International. © TMI 2011, 2012. All rights reserved.

